Dynamic Tissue Systems

REF

CWK08
CWK08-CE
CWK-EX
CWK-H
CWK-3

Instructions For Use

(FR)

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Not made with natural rubber latex.

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STERILE

**UK
CA** 0086

Emergo Consulting (UK) Limited
c/o Cr 360—UL International
Compass House, Vision Park Histon
Cambridge, CB24 9BZ
United Kingdom

MADE IN CANADA

ABRA® Abdominal Wall Closure Set

Abdominal Wall Closure Set is designed to close a retracted full-thickness abdominal wound up to 30 cm long by 30 cm wide.

ABRA® Abdominal Wall Closure Set Extensions

Extension sets are designed to be used with Abdominal Wall Closure Set for wounds longer than 30 cm.

Indications

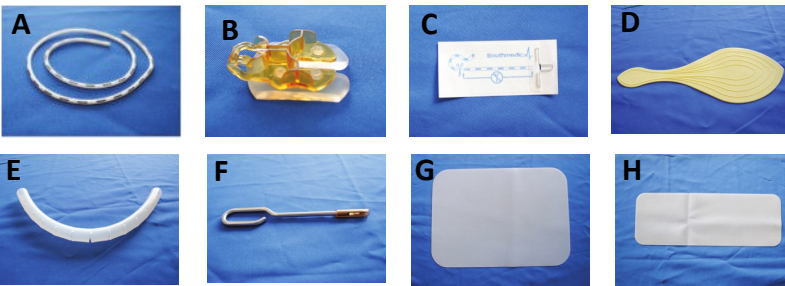
Indicated for use in controlling, reducing or closing retracted soft tissue wounds.

Warnings

Must not be left on for more than 29 days.

Single use product not to be reused due to effect on performance, cross contamination, and risk of infection.

Components:

Note: Components A, B, C, E, F, G, and H are legally manufactured by Southmedic Inc., using radiation as the sterilization method.

Note: Component D is legally manufactured by Greer Medical, Inc., using ethylene oxide as the sterilization method.

CWK08* and CWK08-CE — ABRA® Abdominal Wall Closure Set

A. 50 cm Silicone Elastomer (1/pack)	}	10 packs/set
B. Button Anchors (2/pack)		
C. Button Tails (2/pack)	}	1/set
D. SurgiFish*		
E. 13-hole Elastomer Retainer	}	1/set
F. Cannulator		
G. Southmedic Surgical Retainer (Large) - Silicone	}	1/set
C. Button Tails (for post-operative care)		

*CWK08 and SurgiFish are not CE Marked

CWK-EX — ABRA® Abdominal Wall Closure Set Extension

A. 50 cm Silicone Elastomer (1/pack)	}	3 packs/set
B. Button Anchors (2/pack)		
C. Button Tails (2/pack)	}	1/set
H. Southmedic Surgical Retainer (Small)		

CWK-H — ABRA® Abdominal Wall Closure Set H


A. 50 cm Silicone Elastomer (1/pack)	}	10 packs/set
B. Button Anchors (2/pack)		
C. Button Tails (2/pack)	}	1/set
F. Cannulator		

(Not available for sale in the US)

CWK-3 — ABRA® Abdominal Wall Closure Set Extension (3 Pack)

A. 50 cm Silicone Elastomer (1/pack)	}	3 packs/set
B. Button Anchors (2/pack)		
C. Button Tails (2/pack)	}	

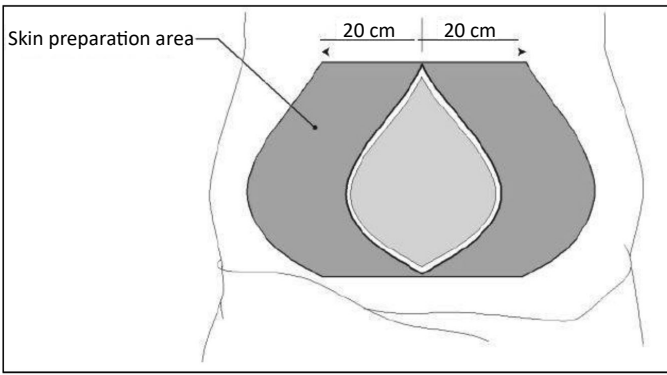
Patient Preparation

Remove:

- Necrotic tissue
- Adhesions
- Inter-layer granulation

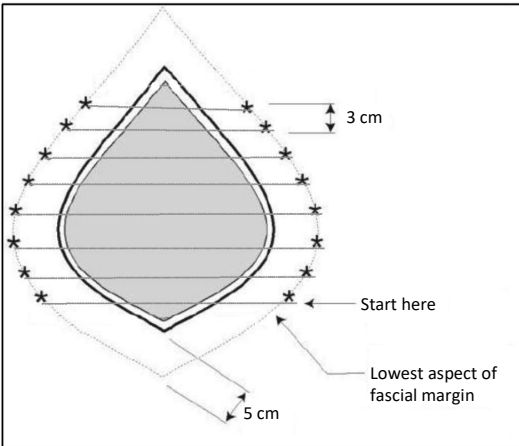Installation

Identify and draw a line 5 cm lateral to the fascial edge of the wound on both sides.

On the 5 cm line, draw hash marks at 3 cm intervals. The opposing marks must be equal and parallel.

Starting at the most inferior hash mark, number sequentially on one side of the wound. Do the same on the contra lateral side to establish matching pairs for Elastomer placement.

Confirm that the most superior and inferior markings ensure Elastomer visualization within the wound.

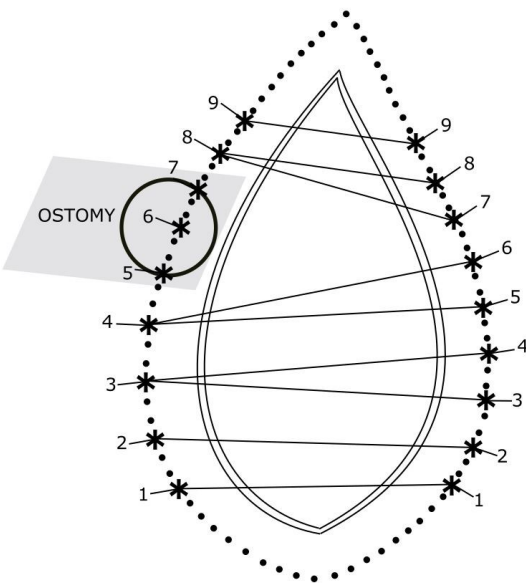
To work around existing appliance, reduce number of Button Anchors on appliance side by 3:

Use the appliance (e.g. ostomy device) as a template, and draw an outline of it at the placement site.


On the 5 cm line, measure 1.75 cm superior and inferior of the ostomy appliance and make initial hash marks.

Beginning at these hash marks, measure at 3 cm intervals both inferior and superior the length of the wound. Continue making the 3 cm hash marks on the contra lateral side. These opposing marks must be equal and parallel.

On the non-appliance side, starting at the most inferior hash mark, number sequentially. Do the same on the ostomy side noting that you will likely skip three of the numbers due to the ostomy appliance.



Make Incisions For Elastomers

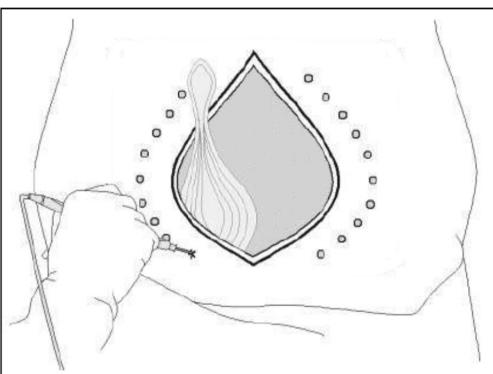Using electrocautery or #11 scalpel blade, make shallow, superficial skin incisions piercing only through the dermal layer at Elastomer placement sites.


Place Incise Drape across entire exposed field and then use a #11 scalpel blade to nick the drape at Elastomer placement sites.

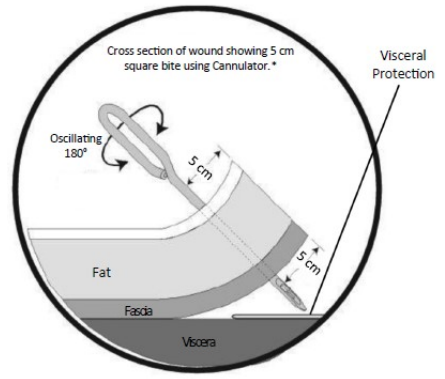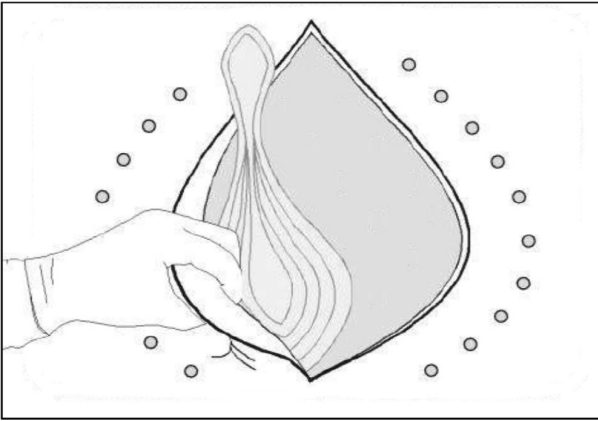Install Elastomers


- Place a visceral protector (e.x. SurgiFish) over the viscera for protection during Elastomer insertion.
- Starting on side of the wound, insert the Cannulator with medium to light pressure using a 180 degree oscillating motion, capturing a 90 degree square bite of tissue.
- Insert and lock 1 cm of Elastomer through the Cannulator loop and pull it back through the tissue. Continue to insert the Elastomers upward along one margin.
- On contra lateral side of the wound, start inferiorly and insert the Cannulator and pull Elastomer through corresponding insertion site.
- Continue sequentially until all Elastomers have been inserted.

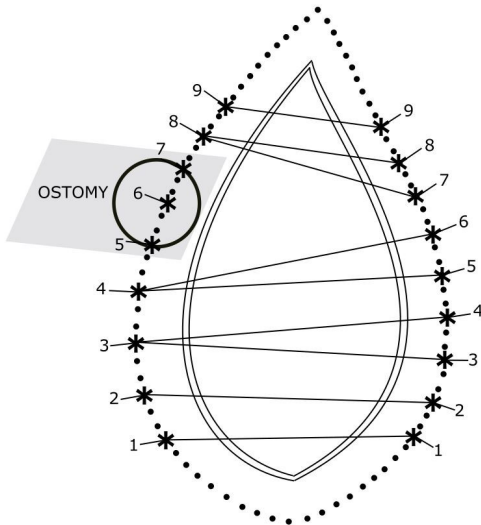Install Elastomers —Work Around Appliance (If Applicable)

- Follow same steps as above, but start on the non-appliance side of the wound first.
- Prior to inserting the Elastomers on the appliance side, map out which Elastomers will be placed 2-to-1 creating a “V-shape” due to the missing Button Anchors.
 - Typically, two of the “V-shapes” will be inferior to the appliance and one superior to the appliance to ensure balanced distribution of force and more support at the midline.
- Start inferiorly, insert the Cannulator and pull Elastomer through corresponding insertion site.
- Continue sequentially keeping track of 2-to-1 placement as previously determined until all Elastomers have been inserted.

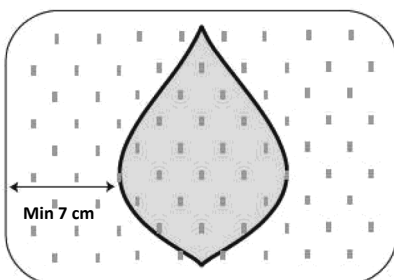Install the Southmedic Surgical Retainer


Southmedic Surgical Retainer Preparation

- Lay the Southmedic Surgical Retainer horizontally across the patient and determine based on the length of the wound if a second Surgical Retainer is required.
 - It should extend a minimum of 2 cm lateral to the Elastomer insertion points *after* tension has been placed and margins have medialized.

Southmedic Surgical Retainer Installation

- Roll the Southmedic Surgical Retainer like a scroll and slide it under the Elastomers. Carefully unroll it and tuck into the paracolic gutters.

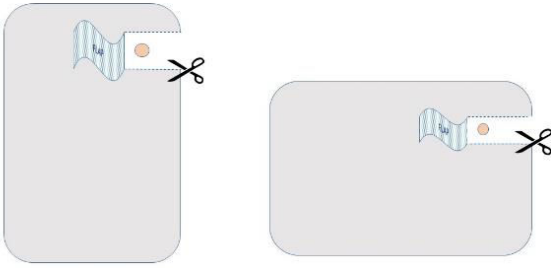Install the Southmedic Surgical Retainer —Work Around Appliance (If Applicable)


When an appliance is present, trim the Southmedic Surgical Retainer as follows:

- Cut two parallel lines approximately 5 to 6 cm apart and approximately 9 cm deep on either side of the appliance creating a flap.
- Fold the flap toward the midline and tuck under the Elastomers for easy access when additional trimming may be required to accommodate progression toward the midline.
- Take time to cut the Surgical Retainer properly to ensure clearance is sufficient while the viscera is protected from the Elastomers.


Install Elastomer Retainer

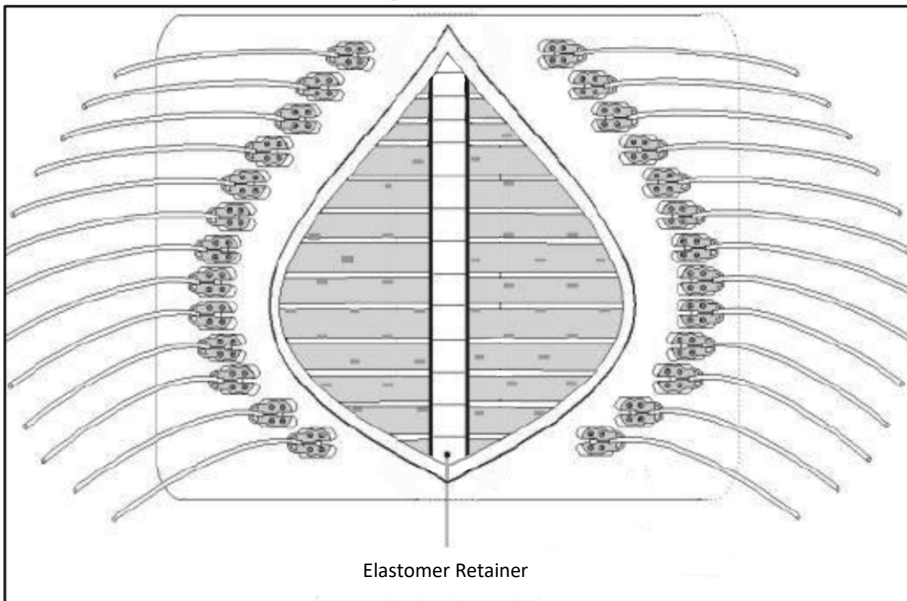Use Elastomer Retainer to elevate Elastomers and prevent migration to upper and lower aspects of wound. Elastomer Retainer can be trimmed to match wound length.


Place Button Anchors

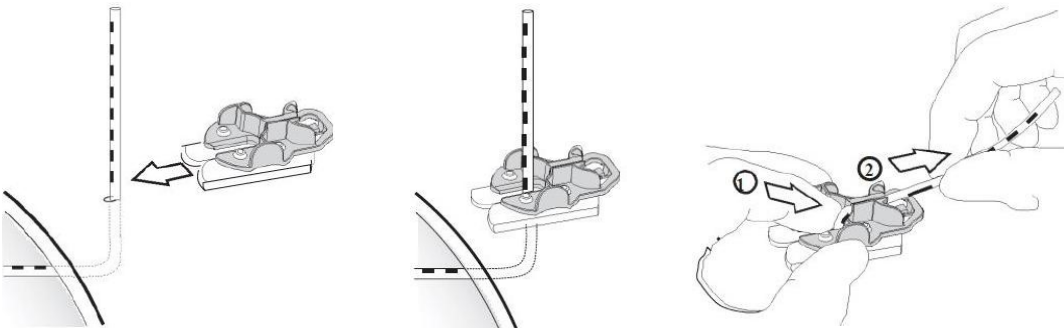Place Button Anchors and secure Elastomer as shown below, but do not tension Elastomer yet:


- Button anchors are placed on the surface of the skin. The silicone pad is designed to adhere to the skin surface without the need of adhesive, sutures, or staples.
- While providing opposing tension to prevent slippage, lift Elastomer upward and slide open side of Button Anchor snug against it. The front of the anchor should be perpendicular to the wound margin.
- Hold Button Anchor firmly in place, and lock down Elastomer at a 90 degree angle. Continue placing Button Anchors sequentially by row.
- Button Anchors should be touching each other, but not overlapping.

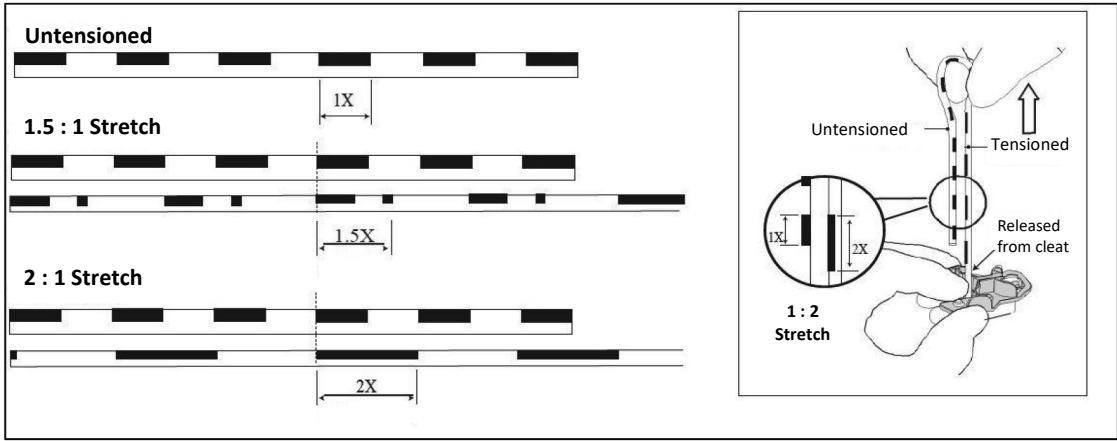Adjust Elastomer Tension

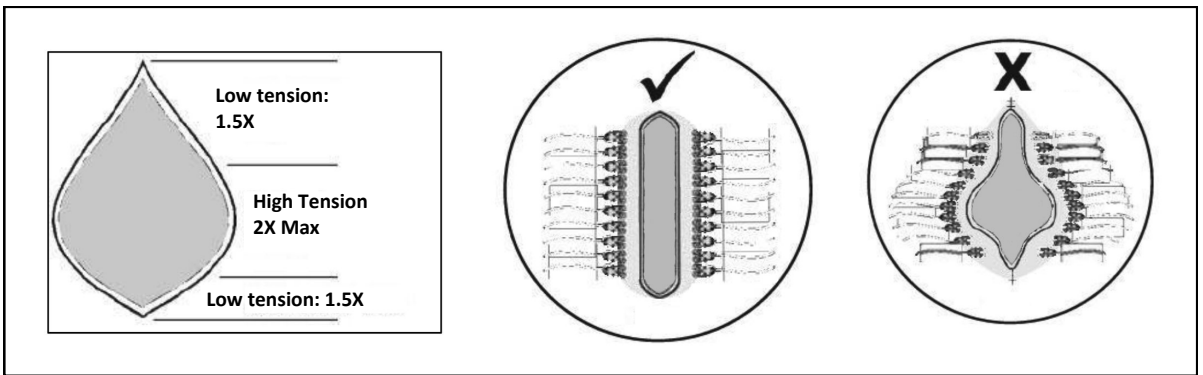Black bars on Elastomer provide visual indication of Elastomer tension. Set tension at 2x stretch in the midline and 1.5x stretch at the inferior and superior aspects of the wound. Use an un-tensioned portion of Elastomer as a guide.
NOTE: If an appliance is present, it is recommended that all Elastomer adjustments are performed from the non-appliance side.



Advance Wound Margins

Advance wound margins by performing “The Move” or the manual medialization of the fascia.

- On both sides of the patient, place two hands lateral to the buttons and rotate fingers outward with palms inward. Firmly press inward and upward simultaneously. Gently rock for 10 seconds and release taking care to avoid any abrupt movements.
- Complete this a total of three times. Then, adjust Elastomers appropriately if reduction in tension has been observed.

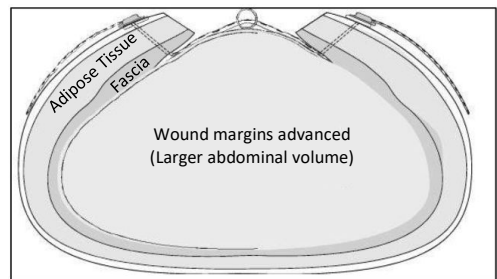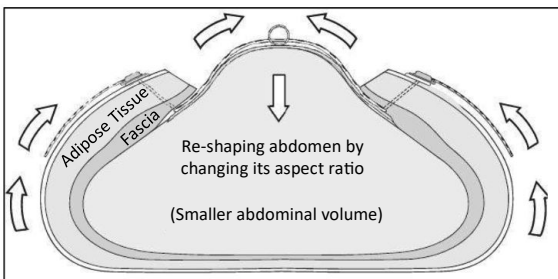The Move

If Clinician Independently Decides To Utilize NPWT

- Follow NPWT manufacturer’s instructions for vacuum application, draw-down and maintenance.
- Note: Do not cover the Button Anchors with the occlusive dressing. Trim and/or tuck the dressing as needed.

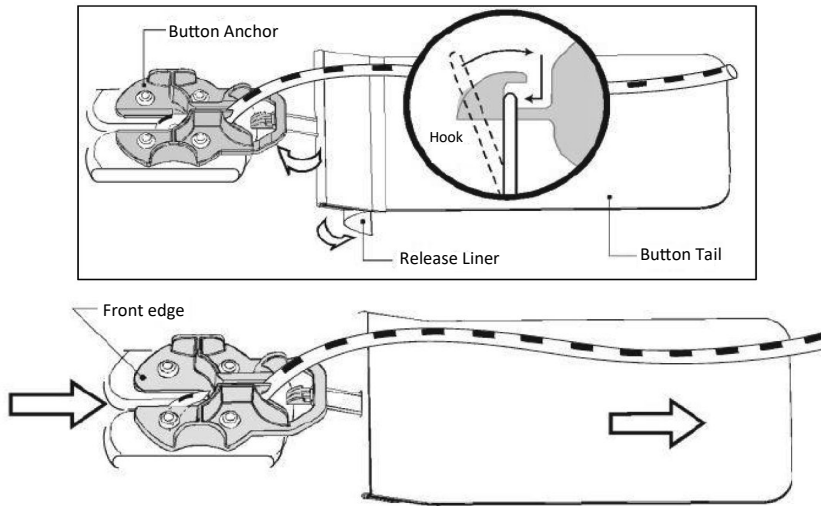
Install Wicking Dressing Under Button Anchors

- It is recommended to install a wicking dressing under each Button Anchor to protect the skin and keep it dry.


Install Button Tails

Button Tails hold Anchors lateral and perpendicular to wound midline.

- Clean and dry skin area 10 cm beyond Button Anchors.
- Fold front of release liner under, without exposing adhesive, then hook Button Tail onto each Button Anchor.
- To fully secure hook to Button Anchor, lift back of Button Anchor and tilt hook vertically to allow it to drop into the slot taking care not to add any additional tension.

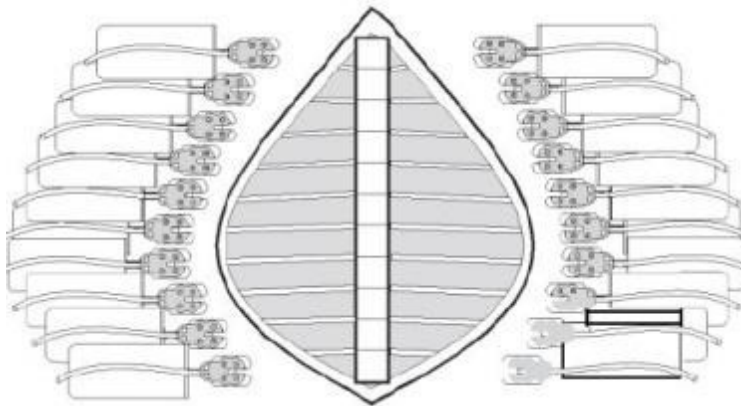- Keeping Button Tail firmly in position, peel off liner and press onto skin, gently rubbing down.
- Button Tails will overlap, but trimming Tails is usually only required to accommodate drains or wounds.
- Button Tails use single-contact adhesive and cannot be re-positioned once applied to skin. Replace as required.

Completed System Installation— Front View

Post-Installation Care

Physician Care

- **The Move:** Perform “The Move” every day—3x for 10 seconds.
- **Elastomer Adjustment (every 2-3 days):** Adjust Elastomer tensions under direct visualization of the black marking. Set tension at 2x stretch at the midline and 1.5x stretch at the inferior and superior aspects of the wound.
- The Surgical Retainer may be trimmed and drawn out of the way as needed. Confirm the Southmed Surgical Retainer is properly providing viscera protection at Elastomer sites and not interfering with appliances.
- If clinician independently decides to utilize NPWT, change NPWT per NPWT manufacturer’s protocol.

Nursing Care

- Continuously monitor under Button Anchors to ensure no moisture is collecting.
- Change the wicking dressing as needed.
- Monitor Button Tails to ensure they are firmly attached. Remove and replace only if they are detached, torn, broken or cleat is exposed.

Closing Wound

When To Close

A low-tension fascial closure can usually be achieved when fascial margins have been re-approximated to within 1 cm. Closing too early may result in unacceptable increases in abdominal compartmental pressure and increased risk of re-herniation. Manually assessing force required to re-approximate fascial margins is recommended at dressing changes and prior to attempting fascial closure.

Instructions for Closing

The Elastomers may be used to keep tension on the wound margins during fascial suture by completing the following steps:

- Prep and drape over the top of the Button Anchors
- Remove any dressings
- Release Elastomers from Elastomer Retainer
- Flip Elastomer Retainer upside down and slide out

Place sutures at either end of wound working toward the center.

- Cut corresponding Elastomers at the midline in progression with the suture.
(NOTE: The weight of the Button will pull the Elastomer outward and away from the wound)

When there is a 5 to 6 cm opening remaining, firmly grab the Southmedic Surgical Retainer with your fingers, a pick-up or something similar. Pull the Surgical Retainer upward, rocking back and forth until it slides out.

Continue suturing until primary closure is achieved.

Critical Points

- If Intra-Abdominal Pressure is adversely increasing during an inflammatory response (as observed by PVP or bladder pressure), Elastomer tensions should be reduced to 1.5x stretch or as required to normalize pressure. This may need to be repeated until inflammatory response subsides.
- Monitor patients fluid balance. If appropriate, consider steps to reduce interstitial volumes. High fluid balance increases abdominal volumes and makes fascial re-approximation more difficult.
- Constipation may increase bowel volume, which may slow wound closure progress.
- Button Anchor pad is occlusive so it's important to clean and dry under buttons at least once every 12 hours, or every nursing shift, to prevent skin maceration. Hydrocolloid or other dressings may be placed under Buttons to assist in moisture collection and load distribution.
- Button Anchors and Button Tails must remain dry. Do not cover with occlusive dressings.
- Buttons Tails must be maintained at all times to avoid skin tears.
- If skin tears are detected due either to excessive Elastomer tension or improper installation of Button Tails:
 - Reduce tension on Elastomer
 - Relocate Button Anchor to original position
 - Re-install Button Tail with adequate tension to hold Button Anchor in position
- If ulcer is detected:
 - Reduce tension on Elastomer at affected Button Anchor
 - Add padding under Button Anchor
- Do not trim Elastomer ends. The added length may be required during inflammatory response period.
- Do not restrain or knot ends of Elastomer beyond cleat of Button Anchor.

During post-operative period, system may be loosened and Southmedic Surgical Retainer repositioned to allow for inspection and cleaning of wound.

During patient transfers, abdominal binders may be used to temporarily support mass of adipose tissues. Do not leave binder in place for > 10 minutes.

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