Instructions For Use

Dynamic Tissue Systems

REF CWM02A

STERILE R

Rx Only

Not made with natural rubber latex.

MADE IN CANADA

50 Alliance Blvd.
Barrie, Ontario
Canada L4M 5K3
1-888-420-1902
Fax 1-705-728-9537
www.southmedic.com

Instructions For Use

ABRA[®] Adhesive Skin Closure Set

Adhesive Skin Set is designed to close skin defects up to 30 cm long with a low to medium force. Topical bedside application.

Indications


Indicated for use in controlling, reducing or closing retracted soft tissue defects.

Warnings

Must not be left on for more than 29 days.
Single use product not to be reused due to effect on performance, cross contamination, and risk of infection.


Adhesive Skin Set

Overview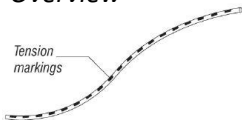

Tension markings

Elastomer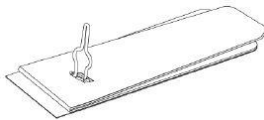

Adhesive Anchor


Wound Bed Preparation

If wound is fresh and margins mobile, skip step.
Otherwise:


- Excise inverted wound margins.
- Remove all granulation tissue.
- Undermine wound edges by 2 cm.

Installation



Mark Skin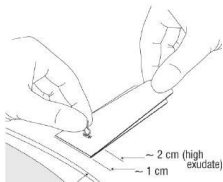


Contour Anchors

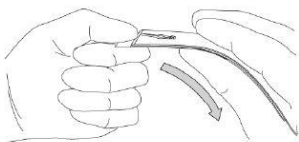
Place Anchors

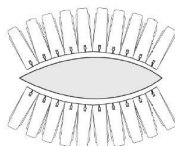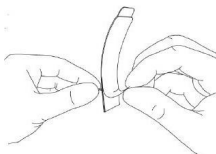Secure Elastomer

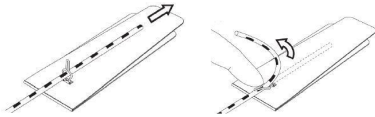
Adjust Elastomer Tension

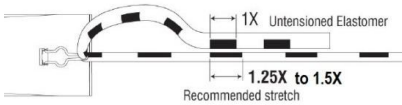
Post Installation Care

Apply Tension to Wound

Reinforcing Adhesive Anchor Installation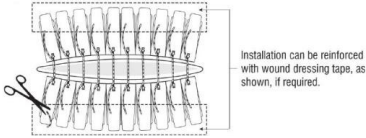

Keeping Adhesive Anchors Dry

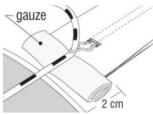Removing Adhesive Anchors

Critical Points

- Therapeutic benefit is derived from cumulative reduction force applied by all Elastomers.
- Over-tensioning any individual Elastomer will result in unnecessary localized trauma creating individual Anchor failure, thereby transferring additional load to remaining Anchors, creating a cascade toward general System failure.
- Wound movement of several centimeters following respiration cycles or body movement is normal with dynamically closing wounds, and indicative of correct overall tension. Movement will reduce as wound approaches closure.
- Do not place any restrictive device on top of System – especially an abdominal binder or pressure stocking – as it will limit the therapeutic effect and may create traumatic pressures on skin.
- Adhesive Anchors must be kept dry to prevent adhesive breakdown.
- Do not restrain or knot Elastomer end behind cleat of any Anchor.
- Skin reactions from the Adhesive Anchors may occur.

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