

ANTI-SPIL™ ADAPTER

REF

9010-F 9010-H
9010-S

Symbol Definitions

MD

Medical Device

Intended Use:

Anaesthetic transfer tube used for the transfer of anaesthetic agents from bottle to vaporizer port

Warnings:

Do not fill the vaporizer with any anaesthetic agent other than the one specified on the vaporizer and on the cap of the device. Use of any other agent than that specified can be dangerous to a patient.

Do not overfill. Overfilling may cause patient injury due to potential overdosing.

Inspect all components for damage or wear before each use. Do not use device if any component is damaged.

Instructions:

The device is designed for use with vaporizers with funnel filler system. Ensure the bottle collar on the bottle matches with the adaptor slots on the device. Line up the adaptor slots to the bottle collar and securely thread adaptor onto bottle. Pour by tilting the bottle. If the device is attached to the bottle after use, ensure the cap is engaged to prevent anaesthetic agent escape to the environment.

Reuse. cleaning:

Clean only the exterior of device with a damp cloth. Do not clean the fluid or gas pathways. Do not immerse the device in any liquid. Do not sterilize the device.

Reporting Suspected Side Effects:

If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at www.southmedic.com.



50 Alliance Blvd.
Barrie, ON
Canada L4M 5K3
1-800-463-7146
southmedic.com

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Rx only

Made in Canada

IFU0024E