**VAPOFIL™**  
Keyed Anesthetic Agent Adaptor

REF 8907-S 8907-H  
8907-F

Symbol Definitions  
MD Medical Device

EC BPP  
Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**Rx only**

**INTENDED USE:**  
Keyed anaesthetic agent adaptor used for the transfer of anaesthetic agents from bottle to vaporizer port

**WARNINGS:**  
Failure to obtain a gas tight seal at both ends of the bottle adaptor may permit leakage of air into the bottle and consequent over-filling of the vaporizer.

The device should not be used if fluid is detected in the outer clear tubing or any components are visibly damaged.

Do not overfill. Overfilling may cause patient injury due to potential overfilling.

**Sight Glass**

**Filling**

Locking Clamp Lever  
Filler Port  
Keyed Vapofil Block End  
Vapofil Bottle End  
Port Valve  
Bottle Collar

## Filling Instructions

1. Remove anesthetic bottle cap & ensure that bottle neck is free of defects. Tightly attach **Vapofil™ Bottle end** to bottle neck and tighten. This connection should be airtight.
2. Check that vaporizer control dial is in "OFF" Position.
3. Hold bottle below the level of the **filler port** and insert **keyed Vapofil™ block end** into **filler port**. Tighten the locking clamp by pulling the **locking clamp lever** forward and downward until the Vapofil™ block end is clamped.  
**Note:** Only the matching block will be allowed to enter the filler port. Ensure the proper Vapofil™ is being used.
4. Raise the bottle above the level of the **filler port** and then open the **port valve** by pulling the lever forward.
5. When the vaporizer is full close the **port valve** by returning the lever to the closed position.
6. Return the bottle to below the level of the **filler port** and allow any excess agent to drain back into the bottle.
7. Release the locking clamp by returning the **locking clamp lever** to the up position and then remove the Vapofil™ block end from the filler port.

**NOTE:** Both the Block end of the Vapofil and the Bottle end of the Vapofil are keyed to fit only the appropriate Vaporizer and Anesthetic Bottle respectively. Please ensure that the appropriate Vapofil is being used. These units are also color coded to again ensure that only the appropriate Vapofil is used.

### Special Note: Perishable Products

Rubber or plastic goods are guaranteed against defects in material or workmanship at the time of delivery.  
**IMPORTANT:** Please note that these instructions were intended as a guideline for use of the Vapofil™. Specific filling and draining instructions may vary depending on the model and type of vaporizer you are using. Please refer to the vaporizer manufacturer's manual for specific instructions on filling and draining your vaporizer.

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**WARNINGS:**  
Failure to obtain a gas tight seal at both ends of the bottle adaptor may permit leakage of air into the bottle and consequent over-filling of the vaporizer.  
The device should not be used if fluid is detected in the outer clear tubing or any components are visibly damaged.  
Do not overfill. Overfilling may cause patient injury due to potential overfilling.

**Draining**

Labels: Locking Clamp Lever, Filler Port, Keyed Vapofil Block End, Port Valve, Sight Glass, Vapofil Bottle End, Bottle Collar.

### Draining Instructions

1. Attach **Vapofil™** to bottle neck. Note that only the matching **Vapofil™** will be allowed to thread onto bottle neck.
2. Insert the **keyed Vapofil™ Block end** into the **filler port**.
3. Tighten the locking clamp by pulling the **locking clamp lever** forward and downward until the **Vapofil™ block end** is clamped and open the **port valve** by pulling the lever forward.
4. Lower the bottle below the level of the **filler port** to allow the vaporizer contents to flow into the bottle.
5. When draining is complete, close the **port valve** by returning the lever to the closed position.
6. Release the **locking Clamp** and remove the **keyed Vapofil block end** from the vaporizer.

**Reuse, Cleaning:** Clean only the exterior of device with a damp cloth. Do not clean the fluid or gas pathways or ports or interior of the cap. Do not immerse the device in any liquid. Do not sterilize the device.

### REPORTING SUSPECTED SIDE EFFECTS

If you suspect that use of this medical device is associated with an adverse occurrence please report the details of the incident to your health authority, and your local distributor or directly to the manufacturer at [www.southmedic.com](http://www.southmedic.com).

**Warranty:** All merchandise is guaranteed to be free from defect in material and workmanship for a period of 30 days from the time of delivery. The merchandise is not covered by any warranty, expressed or implied, including without limitation, any warranty or merchantability or the suitability of the merchandise for a particular purpose. In no event shall Southmedic Inc. be liable for damages of any kind, special, indirect or consequential, howsoever caused. Southmedic Inc. representatives are without authority to charge to foregoing.

Rx only

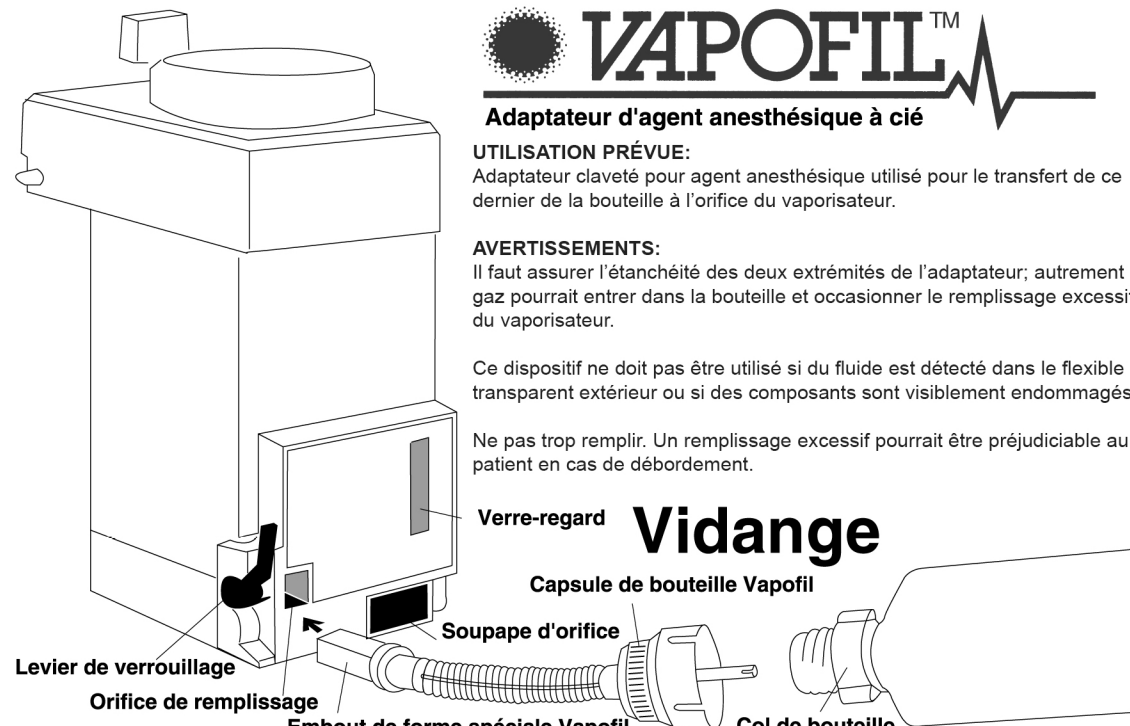
REF 8907-S  
8907-F

8907-H

EC	REF
Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands	

50 Alliance Blvd., Barrie, Ontario, Canada L4M 5K3  
Made in Canada

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