DYNAMIC TISSUE SYSTEMS®

Another **Southmedic**  Solution

DynaClose® DynaStretch®

REF	DWC24	DSX24
	DWC38	DSX38

Instructions for Use

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(SV) Bruksanvisningar

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Note: DynaClose®/DynaStretch® are legally manufactured by Southmedic Inc., using radiation as the sterilization method.

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UK
CA 0086

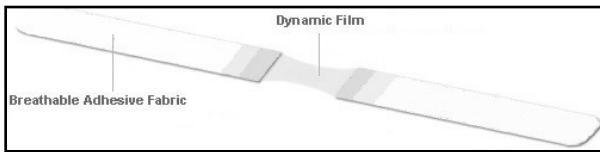
Emergo Consulting (UK) Limited
c/o Cr 360—UL International
Compass House, Vision Park Histon
Cambridge, CB24 9BZ
United Kingdom

MADE IN CANADA

DYNACLOSE STRIPS

Overview


Wound Bed Preparation

If wound is fresh and margins mobile, skip step. Otherwise:
Excise inverted wound margins.
Remove all granulation tissue.


Installation

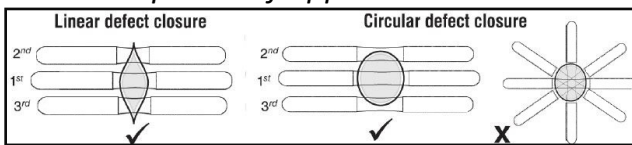Choose Sequence of Application


Ensure skin is clean and dry prior to installation of DynaClose®/ DynaStretch®.

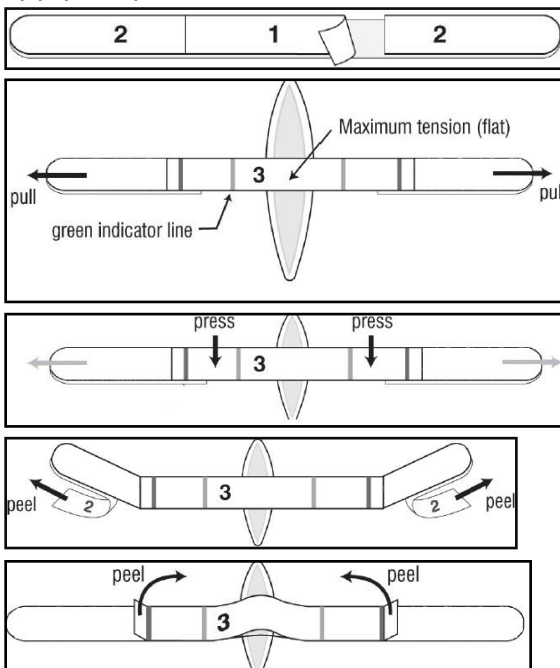Apply Strips

Post-Installation Care

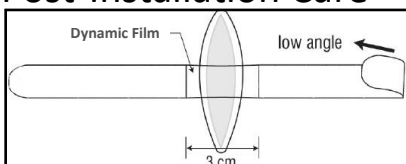Apply as directed. Replace strip when Elastomer tension is reduced to 3.0 cm (for DWC38) and 1.5 cm (for DWC24) in length.

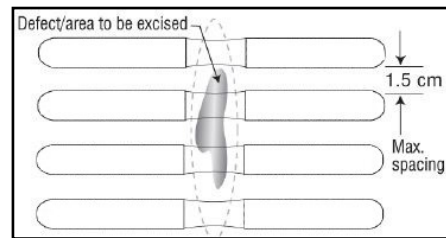
DynaClose® Strips

DynaClose® Strips are designed to close skin defects up to 2.5 cm (DWC24) / 5.0 cm (DWC38) wide with a low force. Topical bedside or home application.


DYNASTRETCH STRIPS

Overview


Layout

Patient-Managed Stretch Phase

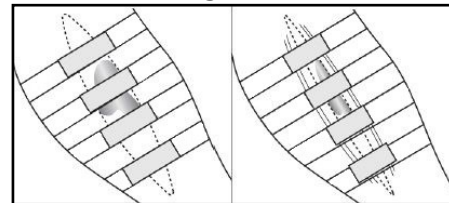Excision & Closure in One Procedure

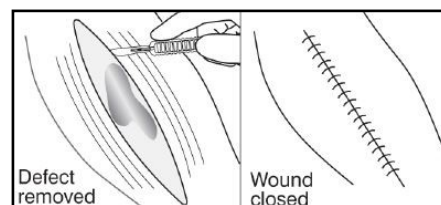Installation

Same as for DynaClose®

Post-Installation Care

Replace as needed to maintain elastomer tension.
Discontinue when sufficient skin depot developed.

DynaStretch® Strips

DynaStretch® Strips are designed to provide external pre-surgical skin expansion over an area of planned excision up to 2.5 cm (DSX24) / 5.0 cm (DSX38) wide. Topical bedside or home application.

Indications

Dynamic Wound Closure Systems are indicated for use in controlling, reducing or closing retracted soft tissue defects. DynaStretch® Strips can be used for pre-surgical skin expansion.

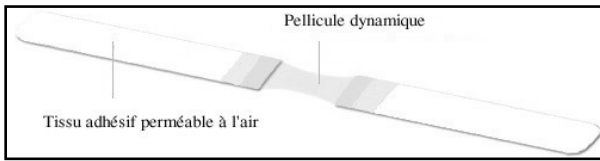
Warnings

Must not be left on for more than 29 days. DynaClose® must not be used on defects wider than fully-stretched elastic film. Do not allow strips to encircle an extremity as this will restrict circulation. Single use product not to be reused due to effect on performance, cross contamination and risk of infection. If allergic reaction occurs, discontinue use.

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