Instructions for Use


DynaCleft®

Pre-surgical Cleft Lip & Palate Approximation


REF

DCX10 - DynaCleft® Unilateral, 1.25" x 4.5" (7 pcs/pouch)

DCX20 - DynaCleft® Bilateral, 1.25" x 3.5" (7 pcs/pouch)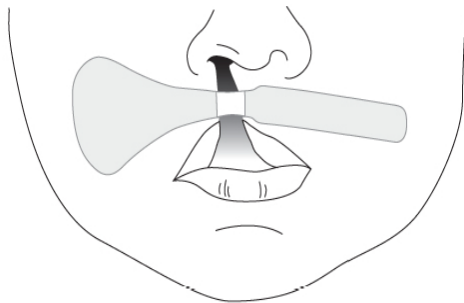

DCX10
Unilateral

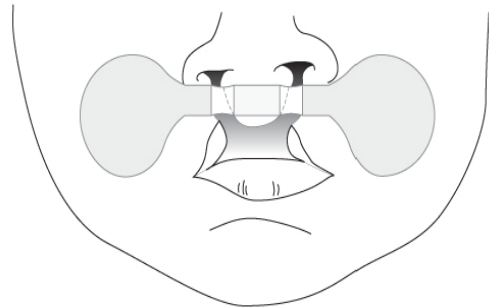DCX20
Bilateral

STERILE R

Rx Only

Not made with natural rubber latex.

50 Alliance Blvd.

Barrie, Ontario

Canada L4M 5K3

1-888-420-1902

Fax 1-705-728-9537

UK
CA 0086

Emergo Consulting (UK) Limited

c/o Cr 360—UL International

Compass House, Vision Park Histon

Cambridge, CB24 9BZ

United Kingdom

To order /pedidos /à l'ordre /Bestellungen /Per ordinare /Om te bestellen/ Encomendas/
För ordering /Για παραγγελία:

1-800-463-7146

+ 1-705-726-9383

Other enquiries /otras consultas /d'autres questions /Sonstige Auskünfte /Altre richieste /
Andere aanvragen /Outros pedidos de informação /Andra frågor /Άλλες έρευνες:

1-800-463-7146

+ 1-705-726-9383

custserv@southmedic.com

www.southmedic.com

About DynaCleft®

Introduction

DynaCleft gently guides soft tissue and bone into a better position for cleft repair surgery under less tension.

Indications

DynaCleft is indicated for use to pre-surgically approximate cleft lip and palate prior to reconstructive surgery. This non-invasive device should be used for as long as possible before baby's first cleft repair surgery to guide lip and palate segments into position for a lower-tension repair.


DynaCleft is available for unilateral and bilateral applications. Typical usage is one strip per day (one pouch per week) for several weeks or as indicated by the referring medical professional.

Warnings

1. If allergic reaction occurs, discontinue use.
2. Exceeding recommended tension may result in skin damage or adhesive failure.
3. When removing DynaCleft from skin, peel outside edges and roll towards middle to prevent damage to skin.
4. **NO REUSE:** Risks of reusing DynaCleft include infection and cross-contamination. DynaCleft uses a single-contact adhesive that is not intended to withstand repeated use.
5. Be aware that DynaCleft can bring the alveolar segments (or upper ridges of the gum) closer together than is desirable, making palate repair more difficult. Please follow your clinician's guidance on how close together the segments should be by the time of surgery (usually no closer than 1/8" or 3mm).

Layout and Instructions


UNILATERAL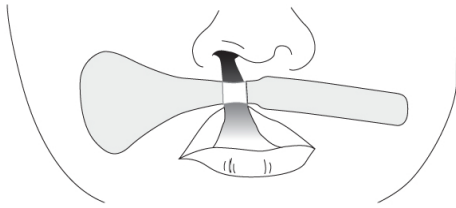

The unilateral cleft will be to one side of the midline of the face. The round end of the DynaCleft should be on the side that has the cleft.


1 Peel Center

Ensure skin is clean and dry prior to installation of DynaCleft. Peel and remove center section of release liner.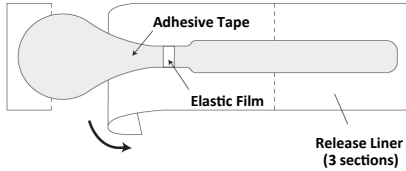


2 Position and Press

Stick the round end of DynaCleft on the skin as shown. Ensure that the edge of the elastic is on the edge of the skin by the cleft.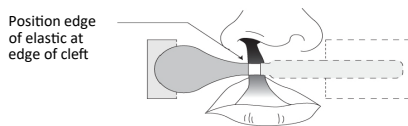

3 Stretch and Apply


Pull the long end of DynaCleft until elastic is 1/4" long (double the unstretched length). With the DynaCleft still tensioned, gently rub the tape into the skin to ensure a good bond.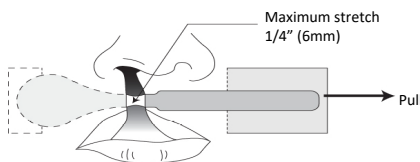

4 Peel Ends and Apply

Peel off outer release liners and complete application to skin. For optimal adhesion, gently rub the tape into the skin. Replace DynaCleft as required to maintain effectiveness.


BILATERAL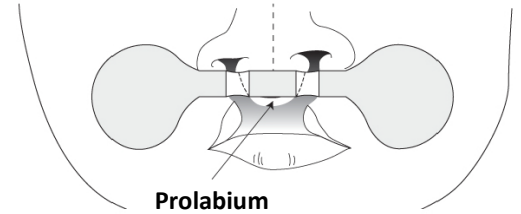

The middle section of the adhesive tape should be adhered as low as possible on the prolabium. Once installed, the rounded ends should be on the cheeks on either side, being careful to keep the prolabium perfectly centered.


1 Peel Center

Ensure skin is clean and dry prior to installation of DynaCleft. Peel and remove center section of release liner.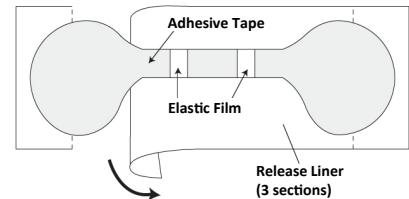


2 Position and Press

Apply the middle section of tape as low as possible on the prolabium as shown.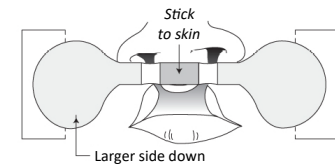

3 Stretch and Apply

Pulling one end at a time, stretch the elastic until it is 1/4" long (double the unstretched length). With the DynaCleft still tensioned, gently rub the tape into the skin to ensure a good bond.

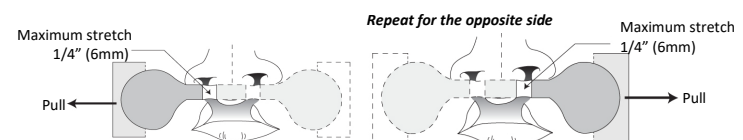4 Peel Ends and Apply

Peel off outer release liners and complete application to skin. For optimal adhesion, gently rub the tape into the skin. Replace DynaCleft as required to maintain effectiveness.

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